**FIRING PROGRAMME - BRENNPROGRAMM
PROGRAMME DE CUISSON - BAKPROGRAMMA
PROGRAMMA DI COTTURA - PROGRAMA DE
COCCIÓN**

Start temperature - Starttemperatur
- Température de départ - 650
Begintemperatuur - Temperatura di
partenza - Temperatura inicial [°C]

Drying time - Trocknungszeit - Temps
de séchage - Droogtijd - Tempo di
asciugatura - Tiempo de secado 02:00
[min:sec]

Closing time - Schließzeit - Temps
d'ajustage - Sluittijd - Tempo di chiusura -
Tiempo de clausura 02:00
[min:sec]

Pre-heating time - Vorheizzeit - Temps
de préchauffe - Voorverwarmtijd - Tempo di
preiscaldamento - Tiempo de precalentamiento 02:00
[min:sec]

Heating rate - Aufheizrate - Vitesse de
montée - Opwarmnelheid - Velocità di
salita - Velocidad de ascensión 980
[°C]

Final temperature - Endtemperatur
- Température finale - Eindtemperatuur -
Temperatura finale - Temperatura final 55
[°C/min]

Final hold - Haltezeit - Stabilisation -
Houdtijd - Stabilizzazione - Estabilización 00:00
[min:sec]

Vacuum - Vakuum - Vide - Vacuüm -
Vuoto - Vacío 100
[%]

Cooling - Kühlphase - Refroidissement -
Afkoeling - Raffreddamento - Enfriamiento -

ITALIANO

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ENGLISH

3-CP is a medical device pursuant to Regulation (EU) 2017/745.

Intended use: Improve metal-ceramic bonding and the final aesthetic result.

Users: For professional dental technician use only. **Contraindications:** Do not use the device outside of its intended use.

Warnings: Perform the processes wearing suitable personal protective equipment. Avoid dust inhalation. **Storage:** Store the product at room temperature in its original, well-sealed container.

Waste disposal: The device is not hazardous and does not require particular waste disposal conditions.

Disclaimer: 3-CP must be used by specialised personnel aware of the technical protocols of use and capable of recognising any device failures. The manufacturer will not be held liable for direct and/or indirect damage resulting from malpractice of the user and/or improper use of the device.

Notice relating to serious incidents: Whenever, during use of the device or in relation to it, a serious incident should occur, the user must report it to the competent authorities of the Member State where the event occurred and to the manufacturer Nobil Metal, specifying the code and lot of the implicated product.

Summary of safety and clinical performance: The summary of the safety and clinical performance (SSCP) for 3-CP can be consulted on the European medical devices database (Eudamed) at the following link: ec.europa.eu/eudamed. The search can be made by means of the basic UDI-DI included in the EU Declaration of Conformity.

INSTRUCTIONS FOR USE

Finish with tungsten carbide burs and sandblast with Al₂O₃ 250 µm; approximately 3.5 bar. Mix 3-CP with distilled water until a smooth paste is obtained.

Apply a unique thin and uniform layer. Proceed to the vacuum firing at a start temperature of 650 °C, increase by 55 °C/min up to the final temperature of 980 °C, without stabilization.

After firing, 3-CP has a light yellow to golden yellow colour, depending on the alloy used.

Proceed with ceramic veneering as usual. **Composition:** Glass ceramic, TiO₂

 **Never re-use already mixed 3-CP.**

The instructions for use, technical data sheets, brochures and safety data sheets (SDS) can be found at www.CKPL.eu.

3 CP
CEKA
PRECI-LINE

Aesthetical conditioner for non-noble metals

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