SHOFU

VINTAGE ZR

06:2019-03
71802

Manufacturer

SHOFU INC.
11 Kamitakamatsu-cho, Fukuine,
Higashiyama-ku, Kyoto 605-0983, Japan
www.shofu.com

EC

REP

SHOFU DENTAL GmbH
Am Brüll 17, 40878 Ratingen, Germany

SHOFU DENTAL CORPORATION
1225 Stone Drive, San Marcos,
California 92078-4059, USA

SHOFU DENTAL ASIA-PACIFIC PTE.LTD.
10 Science Park Road, #03-12,
The Alpha, Science Park II, Singapore 117684

EN

FR

ES

RU

RO

EL

Instructions for use

Mode d'emploi

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SV

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Bruksanvisning

LAYERING DIAGRAM
Uni-layer technique

2-layer technique

3-layer technique

SHADE CHARTS
Basic Shades

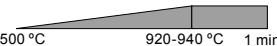
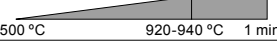
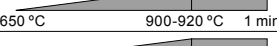
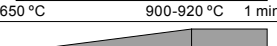
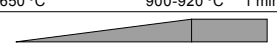
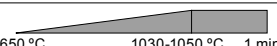
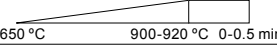
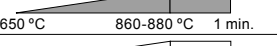
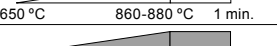
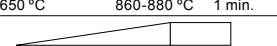
Shade	A1	A2	A3	A3.5	A4	rootA	B1	B2	B3	B4
Opaque Liner	A ₁ O	A ₂ O	A ₃ O	A _{3.5} O	A ₄ O	rootAO	B ₁ O	B ₂ O	B ₃ O	B ₄ O
Cervical	–	–	AC:1 A ₃ B:2	AC:1 A _{3.5} B:1	AC:1 A ₄ B:1	AC	–	–	BC:1 B ₃ B:1	BC
Body	A ₁ B	A ₂ B	A ₃ B	A _{3.5} B	A ₄ B	rootAB	B ₁ B	B ₂ B	B ₃ B	B ₄ B
Opal	57	58	59	59:1 60:1	60	60	57	58	59	60

Shade	C1	C2	C3	C4	D2	D3	D4
Opaque Liner	C ₁ O	C ₂ O	C ₃ O	C ₄ O	D ₂ O	D ₃ O	D ₄ O
Cervical	–	CC:1 C ₂ B:2	CC:1 C ₃ B:1	CC	DC:1 D ₂ B:1	DC:1 D ₃ B:1	DC:2 BC:1
Body	C ₁ B	C ₂ B	C ₃ B	C ₄ B	D ₂ B	D ₃ B	D ₄ B
Opal	58	58	59	60	58	59	59

Whitening Shades

Shade	W0	W1	W2	W3
Opaque Liner	W ₀ O	W ₁ O	W ₂ O	W ₃ O
Cervical	–	–	–	–
Body	W ₀ B	W ₁ B	W ₂ B	W ₃ B
Opal	55	56	56:2 57:1	56:1 57:2

FIRING SCHEDULE

Type of porcelain	Drying time (min.)	Temperature raising speed (°C/min.)	
1 st Opaque Liner	7-8	45	
2 nd Opaque Liner	7-8	45	
Cervical	5-6	45	
1 st Body, Enamel	5-6	45	
2 nd Body, Enamel	5-6	45	
1 st Margin	5-6	45	
2 nd Margin	5-6	45	
Self Glaze	5-6	45	
1 st Gum	5-6	45	
Gum glaze	5-6	45	
1 st Correction	5-6	45	
Correction Glaze	5-6	45	

EN ZR Bond Porcelain

INDICATIONS

High fusing porcelain for fabrication of Zirconia ceramic restorations

PRECAUTIONS

1. If any inflammation or other allergic reactions occur on operator, immediately discontinue use and seek medical advice.
2. Avoid contact with eyes. In case of contact with eyes, immediately flush the eyes with plenty of water and seek medical advice.
3. Wear protective glasses etc while grinding and polishing this product to avoid damages to eyes.
4. Use local dust extractor, dust protective mask, etc while grinding this product to avoid harmful influence of the dust on the human body.
5. This product is intended for use by dental professionals only.

DIRECTIONS FOR USE

Refer to the separate instruction manual and step card.

NOTE ON USE

1. Optimum firing schedule is different depending on the type and model of the furnace. Try to determine the optimal conditions at which every restoration is fired completely.
2. When mixing CLM (Clear Margin) 40 % or more with basic margin porcelain, firing temperature should be lowered. When CLM is used solely, firing temperature should be 980 °C/1796 °F.
3. The Body build-up should be done to the final anatomical shape of the tooth in order to ensure the exact dimension for the Enamel porcelain.
4. Drying time should be altered depending on the size of the restoration to be fired. (Restorations might explode inside the furnace if moisture remains inside.)
5. Enamel porcelain should be built up oversized in consideration of firing shrinkage.
6. When building up additional porcelain onto the dried area, wet the area with water moistened brush in advance.
7. Mix Opaque Liner before use to avoid separation between powder and liquid.
8. Follow the instructions for use of each dental material or instrument to be used in conjunction with this product
9. Do not use this product for any purposes other than specifically outlined in INDICATIONS in these instructions for use.

STORAGE

Store at room temperature(1-30 °C/34-86 °F). Keep away from direct sunlight.

CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.

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